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Dear Customer,

Congratulations on having chosen a quality RUBENS Overbed Table which will certainly fully meet your expectations. We would like to take this opportunity to thank you for showing such confidence in our company and purchasing one of our products.

This instruction manual provides important information regarding safe operation for the patient, user and operator. It will help to learn about all functions of our products and also describes: assembling, care, maintenance of Overbed Table and operating the adjustment mechanism. Read these instructions carefully and follow them exactly in order to provide reliable service and avoid damage or incorrect operation.

We reserve the right to change technical specification of RUBENS Overbed Tables detailed in these instructions, without obligation to update or replace products.

If you have any questions, please do not hesitate to contact your local distributor or *Elbur sp. z o.o. sp.k.* customer service through our website. We will do our best to fulfil your needs.

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1.0. Product description

1.1. Purpose and range of application

RUBENS Overbed Tables are great addition to ELBUR nursing beds type PB offered by our company - *Elbur sp. z o.o. sp.k.* They are intended to alleviate and compensate injuries and disabilities for older, frail and handicapped people. They allow patients to store their private things on them and simplify eating meals, reading books, newspapers or using a computer. They are easy to clean and high quality workmanship.

Our Overbed Tables are made of the best quality material. The surface of steel tubes is coated with a durable polyester powder layer and furniture board is finished with PVC veneer. They stand on four castors – two of them can be locked. RUBENS 3 has got different height adjustment and swivel system than RUBENS 9, what is described in chapter 6.0. Also, RUBENS 9 can be located on both sides of the nursing bed, due to its construction, without loss of functionality, when RUBENS 3 is designed to stand only on the one side of the nursing bed – it has to be specified when placing an order.

Our products meet the varying needs of patients and professional caregivers. In order to meet customer expectations we care about attractive design, as well as matching furniture systems in every place of use. That is why we offer our customers wide choice of decors, although our standard is dark beech décor.

RUBENS Overbed Tables described in this document are classified as Medical Devices Class 1 in accordance with the Medical Device Regulation 2017/745 on medical devices.

1.2. Technical specification

Parameters	Model of Overbed Table	
	<i>RUBENS 3</i>	<i>RUBENS 9</i>
External dimensions [W x L cm]	44,5 x 90,0	40,0 x 79,0
Overbed table tray dimensions [W x L cm]	44,5 x 62,0	40,0 x 58,5
Height adjustment range of the overbed table tray [cm]	72,0 - 106,5	80,0 – 113,5
Angle adjustment range of the overbed table tray [°]	0 – 52,5	0 – 45,0
Castors diameter [cm]	Ø5	Ø5
Safe working load of the overbed table tray [kg]	11,0	20,0
Weight [kg]	14,6	9,8

1.3. Labelling of RUBENS overbed tables

1.3.1. Name plates

Name plate data are defined in the following order:

- model name,
- date of manufacture,
- safe working load of the overbed table tray,
- manufacturer,
- serial number,
- information and warning symbols.

1.3.2. Explanation of symbols used on name plates

- do not use outdoors 
- consult the instruction for use for warnings and precautions 
- follow the instruction for use 
- EC Declaration of Conformity 
- manufacturer 
- safe working load 
- Medical Device 

2.0. Safety information

2.1. Marking and safety signs

Warning

 *Indications of potential risks.*

Note

 *Information and tips regarding the safe use.*

2.2. Potential risks and warning messages

Warnings

-  *Use the Overbed Table in accordance to normal operating procedure and its purpose.*
-  *Before start using the Overbed Table, read instruction carefully and familiarize with rated values provided at name plates, labels and in this manual. It helps to prevent any damage and risk occurring due to incorrect operation.*
-  *Keep the instruction manual for future reference and have access to it at any time – it provides information for operator and patient regarding the convenient handling and safe operation.*
-  *Safe working load of the Overbed Table tray cannot be exceeded. Do not lean on the table – it can cause damage of adjustment mechanism and lead to serious injuries!*
-  *Before every use make sure that the Overbed Table is in proper and faultless condition.*
-  *Only use technically perfect Overbed Table.*
-  *Contact manufacturer or distributor if the exchange or repair of faulty parts have to be made.*
-  *If damage or malfunction is suspected, take the Overbed Table out of service immediately and label it as non-conforming product.*
-  *Use only original Elbur sp. z o.o. sp.k. accessories – modification of Overbed Tables is strictly forbidden without authorization of our company.*
-  *Make sure that other attached equipment can work in a secure and safe manner (do not hesitate to contact manufacturer with questions).*
-  *Make sure that castors have their brakes applied when location of Overbed Table is not going to change.*
-  *Close tray down before relocating the Overbed Table. Do not load Overbed Table tray during pushing it closer to the nursing bed.*
-  *Before adjusting the height of Overbed Table tray, make sure there are not any obstacles, such as furniture, window sill etc.*
-  *Make sure that no cable from electric devices is rolled over during relocating Overbed Table.*
-  *Make sure that no cable from electric devices is damaged, stretched or crushed between Overbed Table parts.*
-  *Do not clean Overbed Tables with a water jet or a high-pressure cleaner. Clean only with a moist towel.*
-  *Never use scouring agents, abrasive cleaning agents, stainless steel cleaners or scouring pads, because they can damage the surface.*
-  *Remove quickly stains, liquid from shelf or top of the Overbed Table – otherwise it can cause dampness of particle board.*

- ⚠ *Read instruction carefully before commencing assembly.*
- ⚠ *Verify the shipment contents with your order before commencing assembly to make sure that you have got all necessary parts and components.*
- ⚠ *Remember to use proper tools during assembly.*
- ⚠ *It is obligated with every maintenance to perform regular inspections, to ensure the safe condition of the Overbed Table.*

The safety information indicated above and found in this instruction manual must always be strictly observed.

3.0. Scope of delivery

Before commencing assembly of the Overbed Table, please read instruction carefully and verify the shipment contents with your order. Please contact your distributor or *Elbur sp. z o.o. sp.k.* customer service, if you miss any component or get something in bad condition

If you place an order for RUBENS 3 you do not to have to assembly Overbed Table, because it is done by manufacturer. That is why you should bring out of the box completely assembled RUBENS 3 Overbed Table.

If you place an order for RUBENS 9 you should expect to receive (numbers in brackets referred to the picture on next page):

- 4 Allen screws M8 x 30 (no. 1 & 3),
- 1 vertical bar (no. 2),
- 1 bottom frame with 4 castors - 2 of them can be locked (no. 4),
- 1 movable support bar with adjustable tray (no.5),
- 1 permanent part of the desktop with fixed angle (no. 6)
- 1 Allen key.

4.0. Spare parts

Spare parts can be ordered from distributor or Elbur company. For reasons of safety, it is allowed to use only original spare parts or accessories from *Elbur sp. z o.o. sp.k.* Otherwise, our company will assume no liability for any accidents, damages, injuries and risks that come about through the use of other spare parts or accessories.

5.0. Assembly of RUBENS 9


Before commencing assembly of the Overbed Table, please read carefully instruction manual and verify the shipment contents with your order (chapter 3.0). Please contact your distributor or *Elbur sp. z o.o. sp.k.* customer service, if you miss any items or get something in bad condition.

Warnings

⚠ *In order to avoid wrong assembly, read carefully instruction manual and pay attention to labelling on the Overbed Table. Remember also to use proper tools.*

Notes

- ⚠ *There must be enough space available for adjusting the Overbed Table tray, its position and height,*
- ⚠ *RUBENS 9 can be located on both sides of the nursing bed, due to its construction, without loss of functionality.*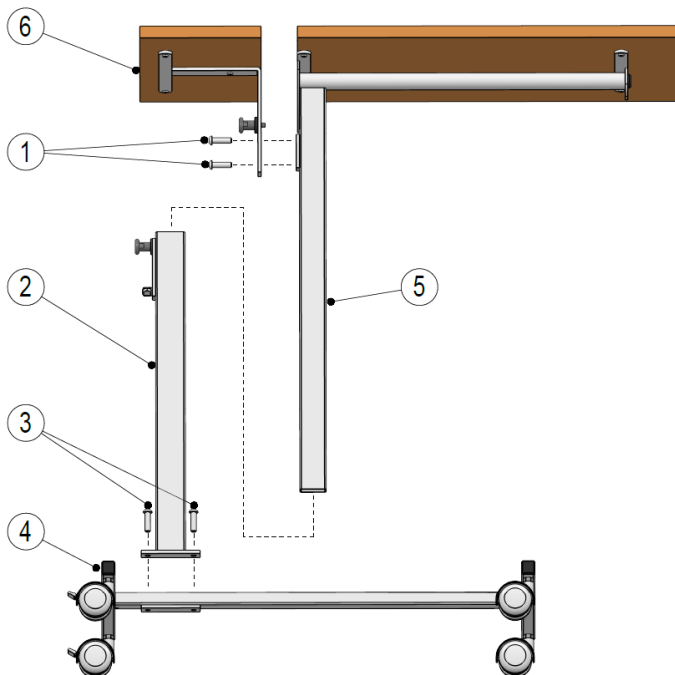

Assembly of RUBENS 9 should be performed in the following order (numbers in brackets referred to the picture attached above):

- 1) carefully bring all components out of the box,
- 2) put bottom frame on four castors (4) on flat, horizontal surface,
- 3) push the castor lever down as far as it goes for the purpose of activating the brake (during the use for the purpose of deactivating the brake push the foot lever toward as far as it goes),
- 4) connect bottom frame (4) with vertical bar (2) using two allen screws (3),
- 5) put movable support (5) into vertical bar (2),
- 6) unscrew 2 allen screws connecting flat spacer to the movable support (5) – they are additionally glued to each other to maintain the correct position.
- 7) attach permanent part of the desktop (6) and its metal angle part to flat spacer of movable support bar, align holes in those parts and connect them using two allen screws (1),

8) read carefully subclause 6.2 to understand how to adjust height of the Overbed Table.

6.0. Using RUBENS Overbed Tables

Warnings

- ⚠ *Do not put your fingers between moving parts – it can caused their entrapment and injuries!*
- ⚠ *Do not load Overbed Table tray before locking it in the proper position and during pushing it closer to the nursing bed.*
- ⚠ *Do not put any drinks or meals on Overbed Table tray when it is swiveled.*
- ⚠ *Safe working load of the Overbed Table tray cannot be exceeded. Do not lean on the table – it can cause damage of adjustment mechanism and lead to serious injuries!*


6.1. Height adjustment and swiveling of the Overbed Table tray – RUBENS 3

Warning

- ⚠ *Before adjusting the height of Overbed Table tray, make sure there are not any obstacles, such as furniture, window sill etc.*

The biggest advantage of the RUBENS 3 is that there is easy possibility to adjust height and angle level of Overbed Table tray.

Press a black releasing button (7), then pull up or press down Overbed Table tray to adjust height in demanded range (release black button when the height is good enough for patient). The adjustment range is between 72,0 and 106,5 cm (8).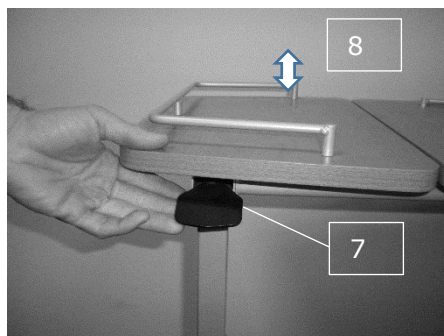

There is adjusting ratchet (9), which allows to swivel Overbed Table tray and lock it into angle position.

In order to raise a tray, pull it up and make sure it clicks into required position before you let it go. There are 5 positions available to achieve for RUBENS 3 Overbed Table tray and the swivel range is between 0° to 52,5°.


In order to lower a tray it is necessary to lift it up all the way to the stopping point, then it can be slowly dropped.

6.2. Height adjustment and swiveling of the Overbed Table tray – RUBENS 9

The biggest advantage of the RUBENS 9 is that there is possibility to adjust height and angle level of Overbed Table tray. Also, RUBENS 9 can be located on both sides of the nursing bed, due to its construction, without loss of functionality.

Unscrew the wing nut (10), pull the pin retainer (11) and movable support up or press it down to adjust height in demanded range (there are 5 positions where the trig can stop and click). Release the pin retainer in the proper position and screw the wing nut back, when the height is good enough for patient. The adjustment range is between 80,0 and 113,5 cm (13).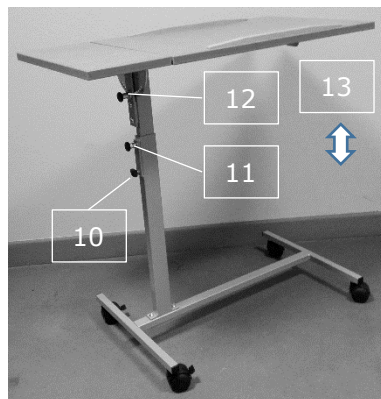

Pull the first pin retainer from the tray (12) and swivel the Overbed Table tray – it clicks into required position after releasing that blockade.

There are 6 positions available to achieve for RUBENS 9 Overbed Table tray and the swivel range is between 0° to 45°.

7.0. Maintenance

Note

- ⚠ *Do not clean Overbed Tables with a water jet or a high-pressure cleaner. Clean only with a moist towel.*
- ⚠ *Never use scouring agents, abrasive cleaning agents, stainless steel cleaners or scouring pads, because they can damage the surface.*
- ⚠ *Remove quickly stains, liquid from the overbed table tray – otherwise it can cause dampness of particle board.*

7.1. Disinfecting

All mild, not aggressive disinfectants set out in EN 12720+A1 can be wiped on the Overbed Tables to clean them. They must be cleaned and washed down with disinfectant in case of a patient change or in other situation required it.

7.2. Cleaning and care

All components of the Overbed Table are made from the best quality material. The surface of steel tubes is coated with a durable polyester powder layer. All wooden or wood-based parts are protected with non-hazardous varnish. The overbed table components can be easily cleaned and maintained according to the effective hygienic rules in specific areas by disinfecting

wiping. Adhering to the described on next page rules of the furniture maintenance ensures its long life and keeps its nice appearance.

CLEANING AND CARE		
NO.	Components	Cleaning method
1.	Steel tubes and coated metal parts	Use a slightly moisturized cloth and normal, mild, not aggressive agents for cleaning furniture
2.	Wooden and wood-based elements	
3.	Plastic components	

8.0. Service

Note

- ⚠ *If damage or malfunction is suspected, take the Overbed Table out of service immediately, label and protect it properly as a non-conforming product.*
- ⚠ *Contact manufacturer or distributor if the exchange or repair of faulty parts have to be made.*

It is obligated to perform regular inspections, to ensure the safe condition of the overbed table:

- First inspection – before first use;
- Service inspection – service should be performed after every repair.

8.1. Troubleshooting

DEFECT	POSSIBLE CAUSES	SOLUTION
Locking castors are not active	Castors unclean	Clean castors, remove all obstacles
	Castors damaged	Notify customer service
There is no possibility to adjust overbed table tray	Gas spring damaged	Notify customer service

9.0. Transport and storage

All ordered Overbed Tables are packed in boxes and placed on the transport palette, which can be relocated by the hand pallet truck or the

forklift. Reliable package protects Overbed Tables from external damages and ensures that patient or user get a Overbed Table in the perfect condition.

Technical specifications (*subclause 1.2.*) contain guidance of operating and storage conditions for Elbur sp. z o.o. sp.k. furniture.

Overbed Tables shall be protected from destruction under external influences before re-storage.

Disassembly shall be performed in the opposite directions to steps specified in paragraph 5.0 – Assembly.

10.0. Disposal

Overbed Tables are made of metal, wooden and plastic components. If they must be scrapped, the plastic and metal parts shall be recycled according to legal regulations. If in doubt, please contact your nearest disposal facility, local community, distributor or Elbur sp. z o.o. sp.k. customer service.

11.0. Warranty

The manufacturer gives a guarantee of 24 months for a perfect functioning Overbed Tables

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